

Professional Rapid Antigen Hpv Test Highly Accurate Rapid Detection Test Cassette

Our Product Introduction

for more products please visit us on oxygenconcentratormedical.com

Basic Information

- Place of Origin: China
- Brand Name: DVOT
- Certification: CE/ISO
- Model Number: HPV
- Minimum Order Quantity: 500 Test
- Price: 0.95USD
- Packaging Details: OPP Packing
- Delivery Time: 15-20 work days
- Payment Terms: D/P, T/T, Western Union, MoneyGram
- Supply Ability: 1000000pcs



Product Specification

- Sample: Female Cervical Swab
- Term Of Validity: 18 Months
- Storage: Stored at 2C-30
- Limit Of Detection: Testing The Limit Enterprise References S Should Be Positive
- Highlight: Highly Accurate Rapid Antigen Hpv Test, Cassette Rapid Antigen Hpv Test, Hpv Rapid Detection Test Cassette



More Images



Product Description

Professional Use Rapid Antigen Hpv Test Rapid Test Kit Rapid Detection Test Cassette accuracy highly

Product Description


The Human Papillomavirus (HPV) Antigen Rapid Test Kit is an in vitro visual read test kit for the qualitative determination of HPV16/18E6 & E7 oncoproteins in cervical swabs from women. It is designed to detect HPV antigens in individuals to aid in the diagnosis of cervical pre-cancer and cancer.



What is HPV virus

HPV virus, known as human papillomavirus, is a genus of papillomavirus vacuolar virus A belonging to the family Papillomaviridae, which is a spherical DNA virus. It causes proliferation of the squamous epithelium of human skin and mucous membranes, and is one of the viruses that have received much attention in recent years

Among them are HPV types 16 and 18, etc. These types of HPV infections are closely related to the occurrence of cervical cancer, oral cancer, rectal cancer, esophageal cancer and other malignant diseases.

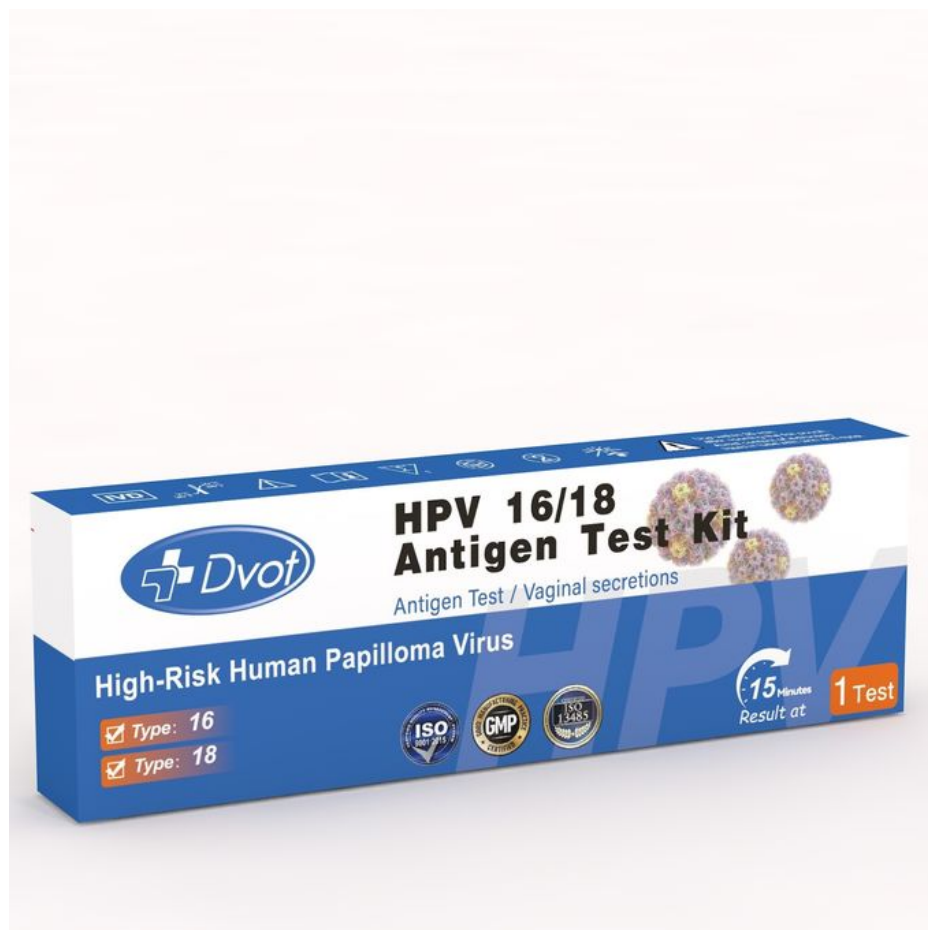

How to use

1. Prepare materials:

Remove all components from the kit, make sure all materials are intact, and check expiration dates according to instructions.

2,Collect samples: Coll ect cericale xloisted cells

3, Sample handling:

Swab the collected sample into a test tube and mix it thoroughly with the solution

4,Waiting for the reaction time:

Cover and shake 10 times

5,Drop in the test card

After dropping the solution onto the test card, wait for 15 minutes to allow the reaction to fully take place.

During this time, do not move or open the bottle as this may affect the test results.

6. Read the result:

Observe whether the card inside the box is open or not, and judge the result according to the "Positive", "Negative" or "Invalid" shown on the card. Negative" or 'Invalid' on the card to determine the test result.

A positive result indicates a possible infection with HPV 16/18; a negative result indicates no infection or detection of either type of HPV; and if the result is invalid, the test needs to be repeated.

*1-04




HPV 16/18 Antigen Test Ki



Human Papilloma Virus (HPV) Antigen Rapid Test Kit is an in vitro, visually read test for the qualitative determination of HPV 16/18 E6&E7 oncoproteins in female cervical swab. This product is intended for detection of HPV antigens from individuals to aid in the diagnosis of Cervical Pre-cancer and Cancer.

- **Sample:** Female cervical swab
- **Term of Validity:** 18 months
- **Storage:** stored at 2°C–30°C
- **Limit of Detection:** Testing the Limit enterprise references S, should be positive
- **Storage:** 2–30°C
- **Shelf life:** 24 months

Test Procedure

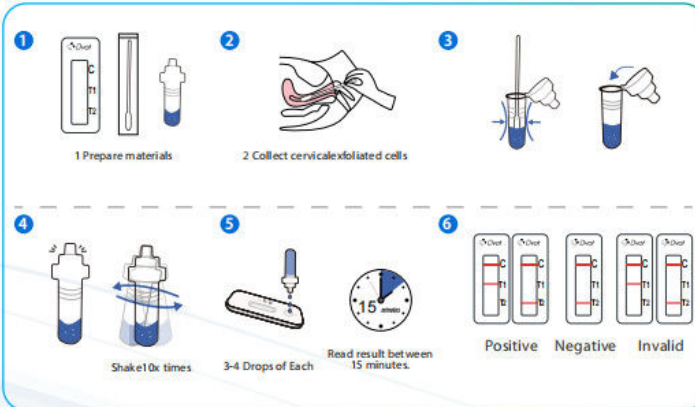

Shenzhen Emeng Health Technology Co., LTD

+86 13760822077

kaylayi0612@foxmail.com

oxygenconcentratormedical.com

Room D, 7th Floor, Guang Long Building, 162 Ping Xin North Road, PingHu Street, LongGang District, ShenZhen